

# En+oli™

## Quick Software Setup Guide



+44 800 088 5341

ENTOLI MEDICAL  
Fairoaks  
Chobham, GU24 8HU



## Setting up the

# Entoli Medical App

Ensure that your system has been set up and commissioned as per the manufacturers' user guides, including any configuration notes from Entoli. This is an essential step before initiating the software setup.

---




+44 800 088 5341

ENTOLI MEDICAL, UNIT D4,  
West Entrance, Fair Oaks Airport,  
Chobham, GU24 8HU

# 1 Activating the Software License

Hit the "Activate License" button, type in your key and click "Activate".

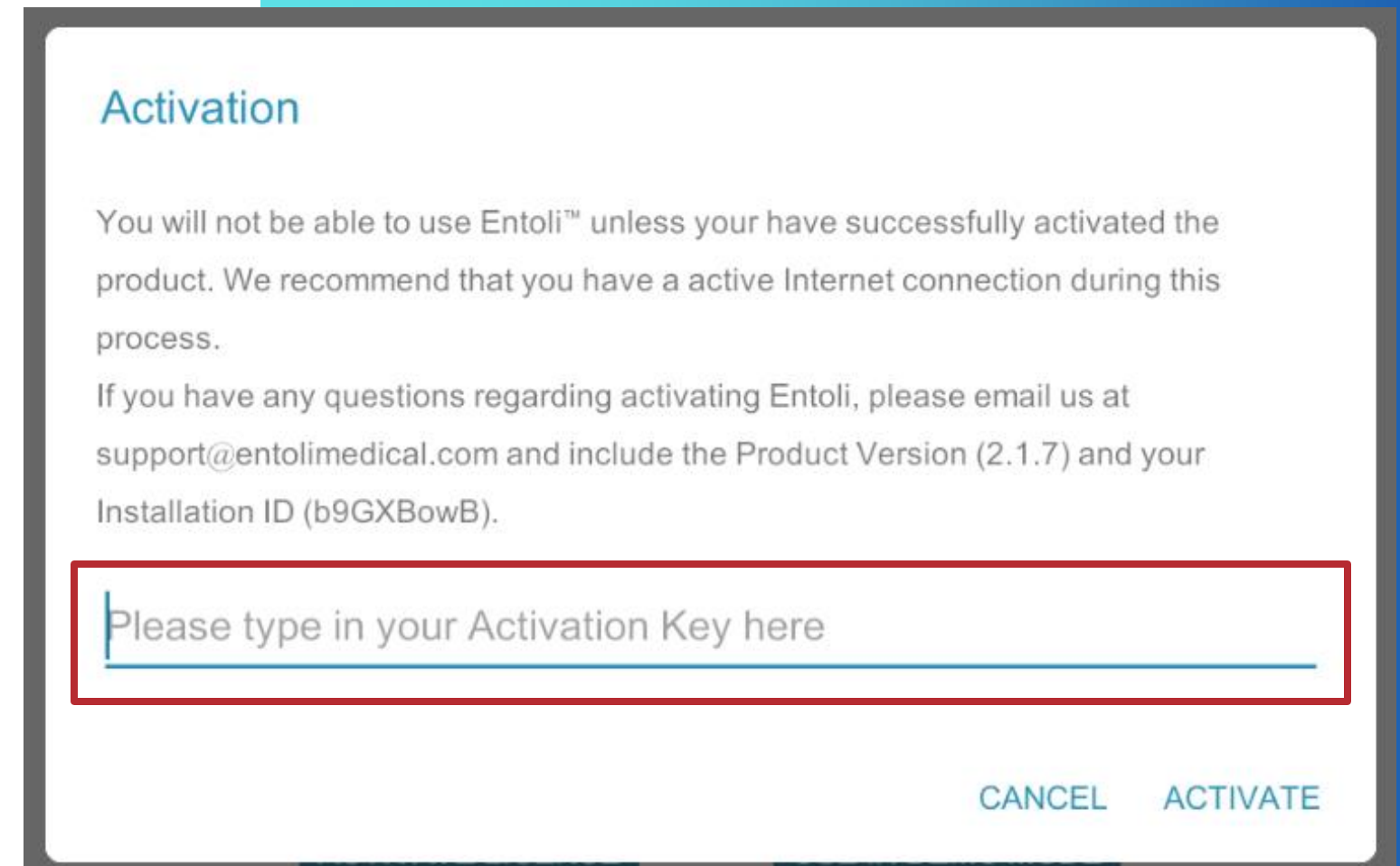

+44 800 088 5341

ENTOLI MEDICAL, UNIT D4,  
West Entrance, Fair Oaks Airport,  
Chobham, GU24 8HU

## 2 Adding Devices

After moving past the license activation, you will be guided through the setup process using a wizard. Add all your devices, which may include the following:



Video Switch



Cameras



Status Camera




Audio Switch



Surgical Lights



Video Recorder

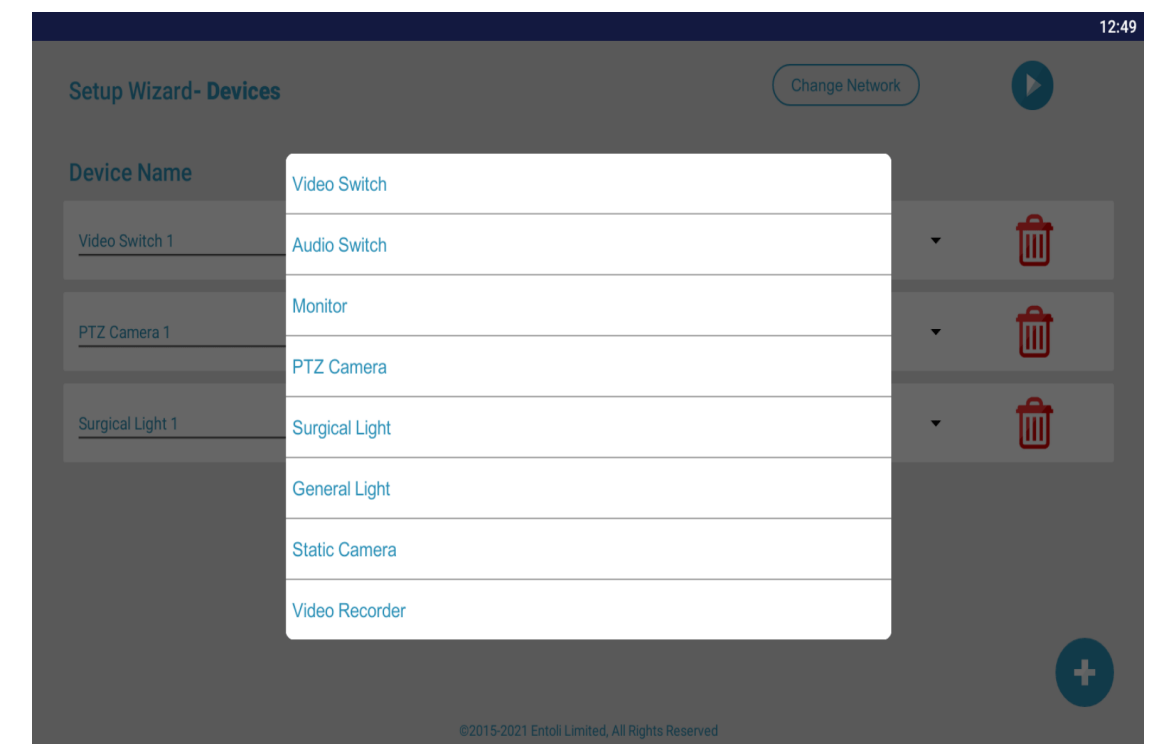

# 3 Labelling and Configuring Devices

Choose the device model from a drop-down menu. We support multiple types and brands, although most of the time it will be the same device type. In this step, you have the option to rename your devices to keep the brand name in the background, if you wish. For example, you can rename the surgical lights to something else than the original default names.

Each device can be labelled according to your preference. This label serves two functions: it is the name that the user will see, and it helps with configuration so you can keep track of what's going on.

The screenshot shows a 'Setup Wizard- Devices' interface. At the top right, there is a 'Change Network' button and a play button. Below is a table with columns for 'Device Name', 'Device Type', and 'Device Model'. Each row represents a device with a trash icon for deletion and a plus icon for adding more devices.

Device Name	Device Type	Device Model
<u>Video Switch 1</u>	Video Switch	Advantech AVAS-212
<u>PTZ Camera 1</u>	PTZ Camera	Sony SRG300
<u>Surgical Light 1</u>	Surgical Light	Rimsa
<u>Static Camera 1</u>	Static Camera	Rimsa Lamp Head
<u>Video Recorder 1</u>	Video Recorder	Medicapture MVR Pro

©2015-2021 Entoli Limited, All Rights Reserved





+44 800 088 5341


ENTOLI MEDICAL, UNIT D4,  
West Entrance, Fair Oaks Airport,  
Chobham, GU24 8HU

# 4 Adding Lamp Heads

Note that a "static camera" here refers to a lamp head camera. Lamp Head One will be connected to Static Camera One. The controls for the camera are sent via the lamp, meaning we have one place to send commands for surgical lights and the same place for cameras.

Setup Wizard- Light Configuration Change Network ◀ ▶

Lamp Name	Camera	
<u>Lamp Head1</u>	Static Camera 1	▼ 
<u>Lamp Head2</u>	Unavailable	▼ 



©2015-2021 Entoli Limited, All Rights Reserved

# 5 Configuring Communication Methods, Protocols, Security and IP Addresses

At this point, you will need to enter any usernames or passwords that you have set up for your devices. Also, provide the IP address assigned to each device.

Setup Wizard- Control Wiring & Comms

Change Network

Device	Commun. Method	Protocol	User Name	Password	IP	Port
Video Switch 1	Direct	HTTP	username123	password123	10.32.32.100	80
PTZ Camera 1	Direct	HTTP			10.32.32.110	52381
Surgical Light 1	Direct	UDP	entoli	.....	10.32.32.106	4001
Video Recorder 1	Direct	HTTP	username123	....	10.32.32.205	3333

©2015-2021 Entoli Limited, All Rights Reserved

For each device, choose the communication method and the protocol:



## Video Switch

Direct communication method, HTTP protocol



## PTZ Camera

UDP protocol (it works better and changes the port).



## Surgical Lights

SSH protocol.



## Video Recorder

HTTP protocol




+44 800 088 5341

ENTOLI MEDICAL, UNIT D4,  
West Entrance, Fairoaks Airport,  
Chobham, GU24 8HU

# 6 Video Wiring

Depending on the devices and the video system used, you will be able to see a certain number of monitors. You can reorder the monitors using the arrows. This step may vary depending on the specific equipment you have selected.

Select a recording output using the toggles. When using the video recorder, for example, you might want to send it to "Output One". Label the inputs and outputs according to your preference as shown below.

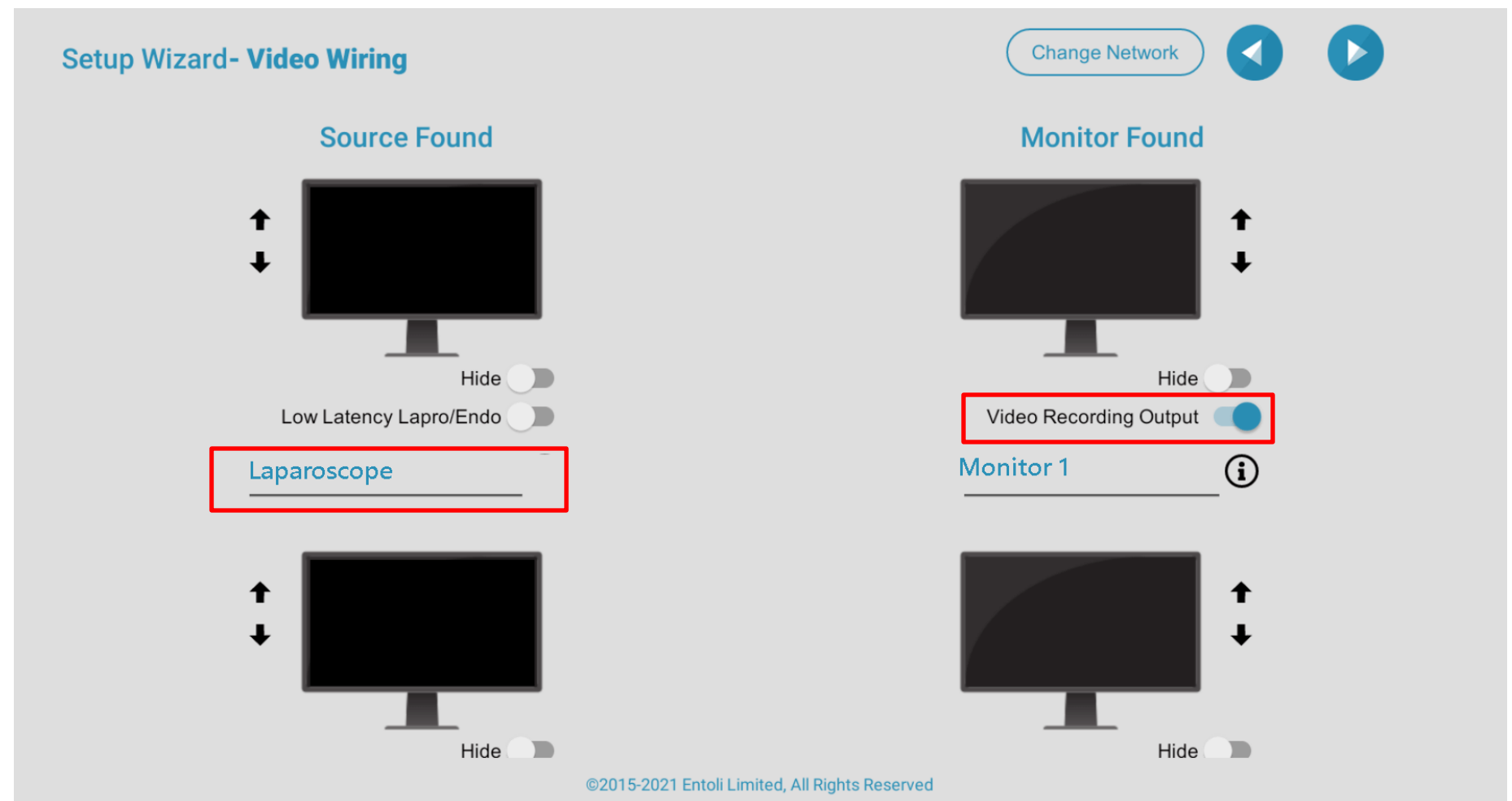



+44 800 088 5341

ENTOLI MEDICAL, UNIT D4,  
West Entrance, Fair Oaks Airport,  
Chobham, GU24 8HU



# 7 System Check & Saving the Configuration

Before moving forward, the app will test if it can connect to these devices. If the test fails, it will not let you progress. Once you are satisfied with the setup, click on the "Save" icon in the top left corner. Give the configuration a name and it will be sent to the cloud. After the setup, you may want to test out the controls for each device. In the event of any issues or conflicts, the system has a logs feature to help.

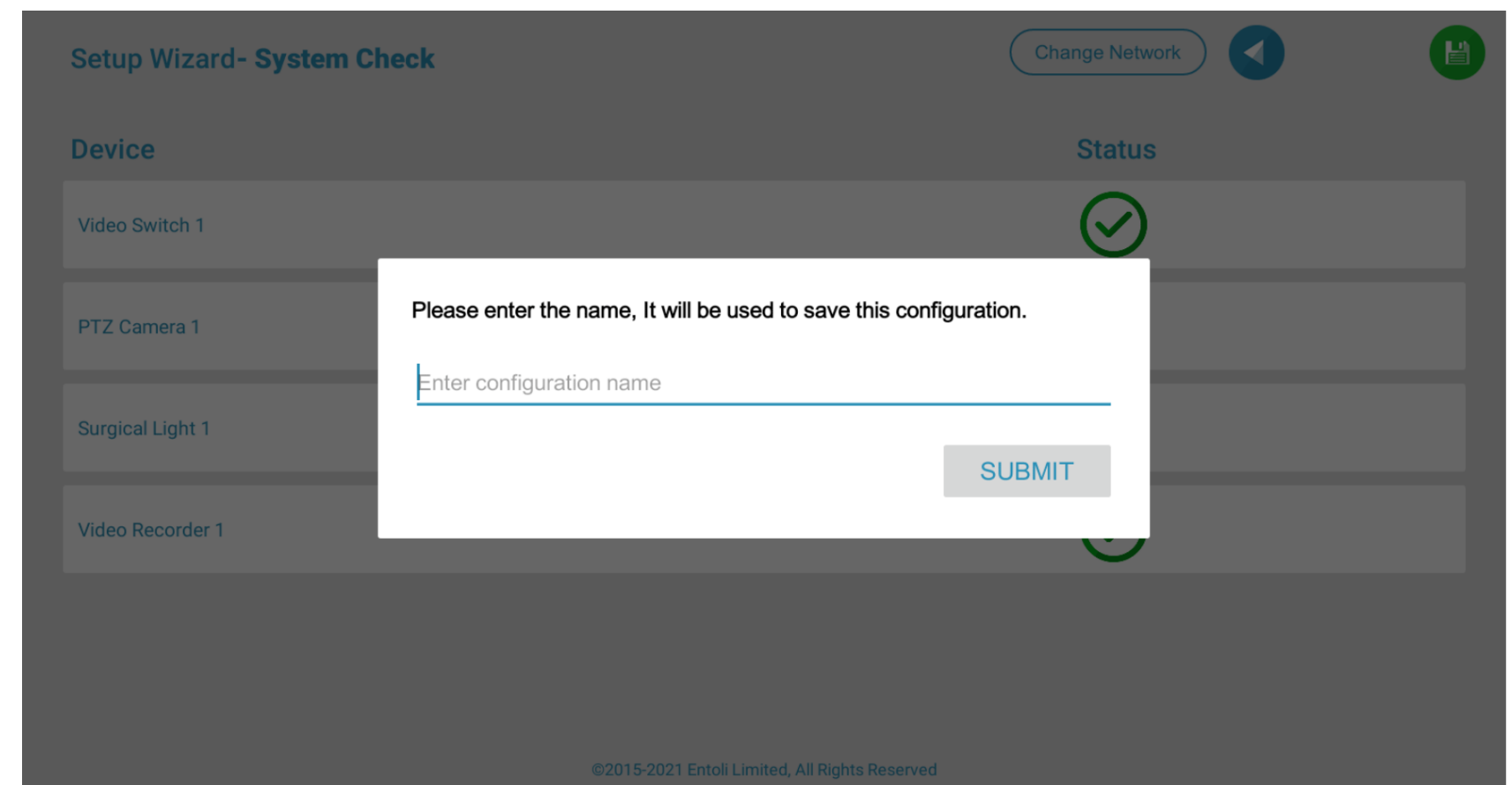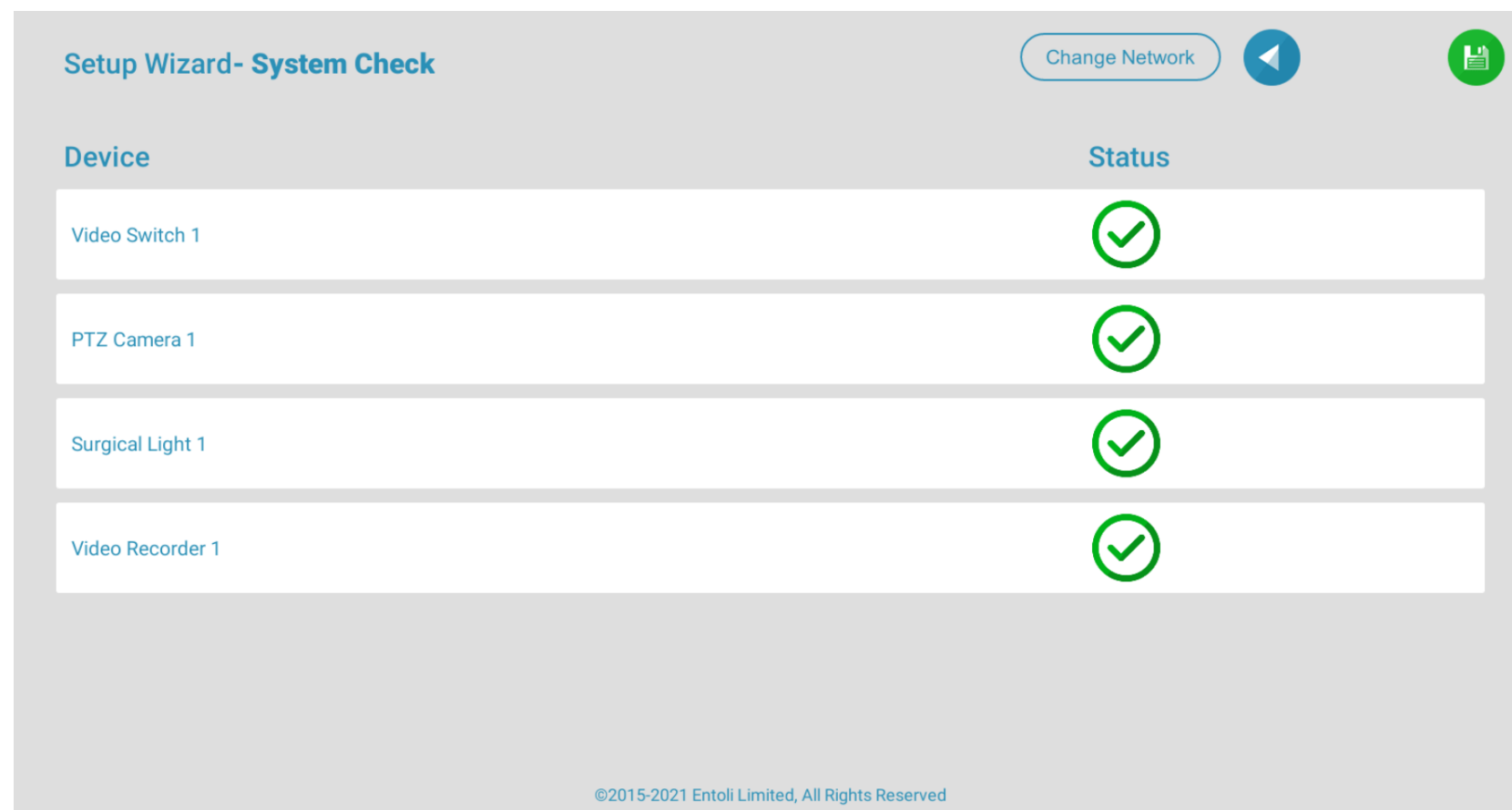



Entoli™

ENTOLI MEDICAL

Fairoaks

Chobham, GU24 8HU

+44 800 088 5341

Operating Theatre Software for the UK,  
Europe, Middle East, Africa, South  
America and the USA.